

Shipping:

- FedEx Overnight shipping, Monday-Thursday only
- Contact information:
 - Alexandra Burne
 - burnea@ufl.edu, 352-294-4071
 - Dr. Mary Brown
 - mbbrown@ufl.edu, 352-294-4029

ELISA testing

- Serum or Plasma, NO whole blood accepted
- Send on Ice Packs
 - Can be sent on dry ice with swabs
- We need a minimum of 1/10mL for testing
 - If sending a small amount please parafilm top to reduce evaporation
- Serum can be stored at -20⁰C
 - 4⁰C Overnight

Culture/PCR Testing for Tortoises

- -We accept:
 - **Choanal (preferred, and best sample)**, Nasal Flush, Swab, and tissues (please email prior to sending tissues).
 - If testing another animal besides tortoise/turtle and/or Tissue, please call in advance to make sure we can test/accept that sample in our laboratory.
 - Please see our “2018 Mycoplasma submission form” for some broad categories of species we offer testing for.
 - Store samples in ultralow until shipping. If ultralow is not available, ship immediately on dry ice. If this is not possible call us for recommendations.
 - **Ship on DRY ICE! This is very important**
 - This will prevent false negatives
 - If sample is warm on arrival we will call to verify if you’d still like the sample to be tested

Culture/PCR Testing for Bovine, Small animal, Lab Animal, Etc.

- 16s rRNA identification is in the event of a positive Culture and PCR isolation.
- Please call/Email: Alexandra Burne or Dr. Mary Brown if you are unsure about collection methods.
- Please provide as much detail on submission form as possible.
 - Call ahead if need be, we are happy to assist!
- **Note:** Bovine testing has a separate submission form.

Shipping Address:

Dr. Mary Brown

University of Florida

Dept. of Infectious Disease and
Immunology

2015 SW 16th Avenue

Rm V2-234